



# secure sleeper™ **vented positioner** *with visco head-support*

## **Positions baby as advised for prevention of SIDS**

A back-only position is advised for healthy, full-term infants by the Surgeon General, the American Academy of Pediatrics and the Sudden Infant Death Syndrome (SIDS) Alliance to reduce the incidence of SIDS.

## **USE INSTRUCTIONS**

Place baby on his/her back onto the soft fabric surface, between the parallel side support wedges (bolsters). Position baby's head on the visco head-support cushion. To properly position the bolsters, snugly secure them against the sides of the baby's torso beneath the armpits. They are designed to fit any baby and should be used to provide added support and comfort as well as keep your baby positioned on his/her back. The visco head-support cushion reduces pressure under baby's delicate head to help prevent Flat Head Syndrome (Plagiocephaly).

## **IMPORTANT SAFETY NOTE**

Discontinue use when baby is able to roll onto his or her tummy without assistance.

## **CARE INSTRUCTIONS**

Remove foam from head cushion. Machine wash cover & bolsters separately in cold water, delicate cycle, no bleach. Drip dry. For best results, wash in a laundry bag.

